INSTRUCTIONS FOR USE

EN

Blephaclean®

This product is a medical device. Please read the instructions before use.

Keep this leaflet. You may need to read it again.

INDICATIONS

BLEPHACLEAN is indicated for the daily hygiene of eyelids.

For use on the skin only, do not put directly into the eyes.

SUITABLE FOR

- Adults,
- Children over 3 months: Parents or carers should administer the product.

CONTRAINDICATIONS

Do not use BLEPHACLEAN if you are allergic to any of its ingredients (see Device Description). In the absence of sufficient data, do not use BLEPHACLEAN, if you are pregnant or breastfeeding unless advised by your doctor.

DEVICE DESCRIPTION

BLEPHACLEAN is a sterile wipe, impregnated with a lotion for the daily hygiene of eyelids.

The lotion contains: capryloyl glycine, sodium hyaluronate, PEG 8, poloxamer 184, polysorbate 20, propylene glycol, hydrocotyl extract (propylene glycol, centella asiatica extract and water), preservative-free Sebum-regulating complex (propylene glycol, Iris florentina root extract, zinc sulphate, alcohol, sodium hydroxide or hydrochloric acid and water), PEG 6 caprylic/capric glycerides (i.e. Macrogol 6 glycerol caprylocaprate), dipotassium phosphate, sodium hyalurate, potassium dihydrogen phosphate, purified water.

The color of the wipe may vary. This is due to natural plant extracts and is normal.

BENEFIT, PERFORMANCE AND MECHANISM OF ACTION

BLEPHACLEAN is a sterile wipe for daily eyelid hygiene especially:

- When you have sensitive skin, eyelids or eyes;
- When you need to remove dust, pollen and discharge from your eyelids. This could be part of your care when you have allergy or infection in your eyes;
- When you have inflammation or infection of the eyelids such as blepharitis, Meibomian gland dysfunction, styes or chalazion. These conditions can cause dry eye symptoms like itchy, gritty, sore and/or red eyes;
- When you need to clean your eyelids before ophthalmic procedures such as therapeutic injections into your eye (intra-vitreous injections), refractive laser eye surgery (LASIK or PRK) and cataract surgery. Because it is well tolerated, BLEPHACLEAN can also be used after surgery, based on the advice from your doctor or ophthalmologist.

BLEPHACLEAN cleans and removes dust, pollen and discharge from the eyelids. BLEPHACLEAN respects the skin around the eyes and its natural bacterial flora.

The product has been tested under ophthalmological control.

GENERAL RECOMMENDATIONS FOR USE

Dosage:

Before eye surgery, use **twice a day** for 5 days or as your doctor or pharmacist tells you to.

For other cases, use **twice a day** in the morning and evening for 3-weeks and then once a day for 3 months or as your doctor or pharmacist tells you to. Some eyelid conditions are chronic and

you may need to continue eyelid hygiene long term.

There is no particular training required to use BLEPHACLEAN.

If you wear contact lenses: you must remove them before using BLEPHACLEAN. Wash hands thoroughly before using the wipes. Tear across the sachet to open (Figure 1) and unfold the wipe inside; do not use any tool to open the sachet.

Figure 1: Opening of the sachet.

Wrap the wipe around the top of your index finger.

Keeping your eye closed, carefully place the wipe on your eyelids and the base of the eyelashes (Figure 2).

Gently massage with small circular movements, removing any crusts, secretions or make-up.

Figure 2: Cleansing your eyelids and eyelashes.

In front of a mirror, gently pull your lower eyelid down with your other hand and then rub your finger with the wipe wrapped around it along the edge of your lower eyelid (Figure 3).

Figure 3: Cleansing of the eyelid margin.

Repeat on the other eye, using a new wipe. There is no need to rinse after using BLEPHACLEAN wipes.

CAUTIONS FOR USE

Check the packaging is intact before first use: use only if the sachet is unopened.

Do not use any wipe more than once, as it may not work as well and may cause contamination. You must contact a healthcare professional if your symptoms get worse or do not improve.

WARNINGS

Keep out of the sight and reach of children.

Avoid getting BLEPHACLEAN into the eyes; you may experience a burning sensation. If this happens, rinse with sterile water or saline solution.

SIDE EFFECTS

The following events have been reported with BLEPHACLEAN: burning or itching sensation for some minutes after use, redness or swelling of the eyelids, ocular redness, allergic reactions. In such cases, contact lens wearers should not put their contact lenses back in.

If you note any side effects or unusual sensations after using this product, please contact a healthcare professional and report the information to the local distributor, to the manufacturer or to your local Health Authority (see contacts at the end of the instructions for use).

STORAGE AND DISPOSAL CONDITIONS

Store between 8°C and 25°C.

Do not expose your product to direct sunlight or humidity.

Throw away the wipe immediately after use.

Do not use after the expiry date clearly marked on the sachet and outer carton.

The expiry date refers to the last day of that month as long as the packaging is intact and it has been stored correctly.

Dispose of the product responsibly and in accordance with local guidance (the sachet should be disposed of as household waste, the outer carton and the leaflet are recyclable).

Date of issue and version of the instruction for use: Ver. 3 12/2021

Manufacturer:

Laboratoires Théa
12 rue Louis Blériot
63017 Clermont-Ferrand
Cedex 2 France

www.laboratoires-thea.com

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Device Identifier: BLEPHACLEAN 1179

Distributor: Thea Pharma Inc., 2150 Winston Park Drive, Units 4 & 5, Oakville, ON L6H 5V1

Health authority: Medical Devices Directorate, Health Products and Food Branch, 11 Holland Avenue, Tower A, 2nd Floor, Address Locator: 3002A, Ottawa, ON K1A 0K9

E-mail:

hc.meddevices-instrumentsmed.sc@canada.ca

Telephone: 613-957-4786

MEANING OF THE PICTOGRAMS ON THE PACKAGING

	Manufacturer		Batch number
	Date of manufacture		Use-by date
	Temperature limits		Sterilized using irradiation
	Medical Device		Consult instruction for use
	Single use		Do not use if package is damaged
	Caution		

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Distributeur : Thea Pharma Inc., 2150 Winston Park Drive, Units 4 & 5, Oakville, ON L6H 5V1