EN

THEAHYABAK™

This product is a medical device. Please read the instructions before use.

Keep this leaflet. You may need to read it again.

Dry & tired eyes

Preservative-free & phosphate-free

Suitable for contact lens wearers

1. INDICATIONS

THEAHYABAK is intended to moisten and lubricate the eye and contact lenses.
Ocular use only.

2. SUITABLE FOR

- Adults
- Children: parents or carers should administer the product
- Pregnant or breastfeeding women based on current knowledge
- Wearers of any type of contact lenses.

3. CONTRAINDICATIONS

Do not use **THEAHYABAK** if you are allergic to any of its ingredients (see device description).

4. DEVICE DESCRIPTION

THEAHYABAK is an ophthalmic solution composed of hyaluronic acid (as sodium hyaluronate 0.15%), sodium chloride, trometamol, hydrochloric acid, and water for injections.

THEAHYABAK is a sterile, phosphate-free and pH neutral solution. The solution is supplied in the ABAK® multi-dose bottle. This innovative and patented device provides eye drops through a 0.2 µm filter, preventing any bacterial contamination of the solution without the use of any preservatives.

Therefore, **THEAHYABAK** is gentle for the eyes as it is preservative-free. The absence of preservatives allows the ocular tissue to be protected and does not cause reactions in users who are sensitive to preservatives.

5. BENEFIT, PERFORMANCE AND MECHANISM OF ACTION

THEAHYABAK is recommended for moistening and lubricating the eye in case of dryness or tired eyes caused by external factors such as wind, smoke, pollution, dust, dry heat, air conditioning, air travel, or prolonged work in front of a computer screen. It provides immediate relief and makes the eyes feel more comfortable during the day.

THEAHYABAK is also recommended for contact lens wearers. It lubricates and hydrates the lens, facilitating placement or removal of the lens, as well as allowing them to be worn all day long.

The main ingredient of **THEAHYABAK** is hyaluronic acid (as sodium hyaluronate), a natural polysaccharide found in the human eye which retains water to hydrate and lubricate the surface of the eye. It keeps the solution on the surface of the eye, giving long lasting relief and reducing the healing time of the surface of the eye.

6. GENERAL RECOMMENDATIONS FOR USE

There is no particular training required to use **THEAHYABAK**.

Wash hands thoroughly before use.

Do not use any tool to open the bottle.

When you first use **THEAHYABAK**, you will find the first drop takes longer to come out due to the unique filtration system in the bottle (see device description).

Replace the bottle cap after use.

To moisten and lubricate the eye:

Dosage: 1 drop in each eye, as often as necessary during the day. Apply one drop to the inside of the lower eyelid, while gently pulling the lower lid down and looking up. Close the eye gently after application.

To lubricate and hydrate the lens:

Dosage: 1 drop on each lens when putting them on or removing them, as well as often as necessary during the day. Before placing your contact lens, simply apply a drop into the inside of the lens when it is on your finger. Put your lens in as usual and blink a few times to allow the solution to settle, ensuring an even distribution of **THEAHYABAK** over the eye and the surface of the contact lenses.

7. CAUTIONS FOR USE

Multiple use product for single patient.

Do not share bottles with other people to prevent cross contamination.

Check that the packaging is intact before first using this product. Check that the tamper-proof ring on the bottle is intact before using it for the first time.

To avoid contamination, do not allow the tip of the bottle to touch anything.

Contact a healthcare professional if your symptoms get worse or do not improve.

8. WARNINGS

Keep out of the sight and reach of children.

Do not inject and do not swallow.

After use, do not drive or use machines if your vision is blurred: wait for a few minutes until your vision recovers.

Wait at least 10 minutes before using different eye products.

Never dilute **THEAHYABAK** or mix it with other solutions.

If exposed to low atmospheric pressure, the bottle may leak after the first use.

9. SIDE EFFECTS

The following side effects have been reported: rare possibility of eye irritation, allergic reactions, eye pain, foreign-body sensation, blurred vision and ocular redness. In such cases, contact lens wearers should remove their contact lenses.

When you use eye drops, you might experience disturbing symptoms such as burning sensation in the eye or stinging sensation in the eye for a short time.

If you note any side effects or unusual sensations after using this product, please contact a healthcare professional and report the information to the local distributor, the manufacturer, or your local Health Authority (see contact details at the end of the instructions for use).

10. STORAGE AND DISPOSAL CONDITIONS

Store between 8°C and 30°C.

Do not expose your product to direct sunlight or humidity.

Do not use after the expiry date, given on the bottle and on the outer box.

The expiry date refers to the last day of that month as long as the packaging is intact and has been stored correctly.

The product can be used for 6 months after it has first been opened. After this time, discard any leftover product responsibly.

Dispose of the product responsibly and in accordance with local guidance (the bottle should be disposed of as household waste; the outer carton and the leaflet are recyclable).

Date of issue and version number of the instructions for use: Ver.5 11/2023 (CA)

THEAHYABAK is a trademark of Laboratoires Théa.

©2024, Laboratoires Théa. All rights reserved.

Manufacturer:

 Laboratoires Théa
12 rue Louis Blériot
63017 Clermont-Ferrand Cedex 2 France

www.laboratoires-thea.com

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CA

Importer:

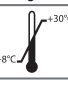
Thea Pharma Inc.
10 Four Seasons Place – Suite 802
Toronto ON
Canada, M9B 6H7

Health authorities:

Medical Devices Directorate
Health Products and Food Branch
11 Holland Avenue, Tower A 2nd Floor
Address Locator: 3002A, Ottawa, Ontario
K1A 0K9
E-mail: meddevices-instrumentsmed@hc-sc.gc.ca
Telephone: 613-957-7285

Device identifier: THEAHYABAK 1557

11. MEANING OF THE PICTOGRAMS ON THE PACKAGING

	Manufacturer		Consult instruction for use
	Batch code		Single patient multiple use
	Date of manufacture		Medical Device
	Keep dry		Caution
	Temperature limits		Keep away from sunlight
	Use by date		Single sterile barrier system with protective packaging outside
	Sterilized using aseptic processing techniques		Catalogue number
	Do not use if package is damaged and consult instructions for use		

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**Importateur :**Thea Pharma Inc.