# THEALOZ® GEL

This product is a medical device.  
Please read the instructions before use.  
Keep this leaflet. You may need to read it again.

## 1. INDICATIONS

**THEALOZ GEL** is indicated to protect, hydrate and lubricate the eye for treatment of dry eye syndrome.

Ocular use only.

## 2. SUITABLE FOR

- Adults,
- Children: Parents or carers should administer the product,
- Pregnant or breastfeeding women based on current knowledge.

## 3. CONTRAINDICATIONS

Do not use **THEALOZ GEL** if you are allergic to any of its ingredients (see Device description).

## 4. DEVICE DESCRIPTION

**THEALOZ GEL** is an ophthalmic gel composed of trehalose (3 %), hyaluronic acid (as sodium hyaluronate salt, 0.15 %), carbomer (Carbopol 974P, 0.25 %), sorbitol, sodium hydroxide and water for injections.

**THEALOZ GEL** is a sterile, phosphate-free, hypotonic and pH neutral fluid gel.

**THEALOZ GEL** is supplied in an easy-to-use multi-dose bottle. This preservative-free system maintains sterility of the gel and prevents bacterial contamination over the duration of treatment.

**THEALOZ GEL** is gentle for the eyes as it is preservative-free. The absence of preservatives allows the protection of ocular tissues and also causes no reactions to users who are sensitive to preservatives.

## 5. BENEFIT, PERFORMANCE AND MECHANISM OF ACTION

**THEALOZ GEL** is recommended in cases of discomfort, sensation of stinging, foreign body or irritation of the eye. These symptoms can be caused by:

- outside factors such as wind, smoke, pollution, dust, sunny or cold conditions, dry heat, air conditioning, plane journeys, long hours working in front of a computer screen;
- eye surgery such as refractive and cataract surgeries.

**THEALOZ GEL** provides comfort to the eyes, prolonged relief from the symptoms of dry eyes. It helps repair the ocular surface damage and relieves ocular surface discomfort occurring after eye surgery. The main ingredients of **THEALOZ GEL** are trehalose, sodium hyaluronate, and carbomer (Carbopol 974P).

Trehalose is a natural substance present in many plants and animals, which can survive in extremely dry conditions. It provides protective and hydrating properties.

Sodium hyaluronate, naturally found in the human eye, holds water to hydrate and lubricate the surface of the eye. It keeps the gel on the eye surface giving prolonged relief and cutting down the healing time of the ocular surface.

Carbomer allows **THEALOZ GEL** to stay on the eye surface longer, providing prolonged relief of dry eye signs and symptoms.

The unique combination of trehalose, sodium hyaluronate and carbomer in **THEALOZ GEL** offers especially prolonged protection, hydration and lubrication of the eye surface.

## 6. GENERAL RECOMMENDATIONS FOR USE

**Dosage:** 1 drop in each eye, from 1 to 4 times a day.

However, as it is a gel, **THEALOZ GEL** is particularly suitable for use in the evening, just before going to bed.

There is no particular training required to use **THEALOZ GEL**.


If you wear contact lenses: you must remove them before using **THEALOZ GEL**.

After use wait at least 30 minutes before putting your lenses back in. Wash hands thoroughly before use.

Do not use any tool to open the bottle.

When using for the first time, before delivering a drop to the eye, you should first of all practise using the dropper bottle by squeezing it slowly to deliver one drop away from the eye.

When you are confident that you can deliver one drop at a time, choose the position that you find most comfortable for the instillation of the drops (you can sit down, lie on your back, or stand in front of a mirror).

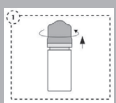- 1) Rotate the cap counter-clockwise to break the seal. To avoid contamination, do not touch the tip of the bottle with anything.

- 2) Apply one drop in the pocket of the eyelid by squeezing the bottle, while gently pulling the lower lid down and looking up. Please note there might be a few seconds delay between squeezing and the drop coming out. Do not squeeze too hard. Close the eye gently after application. Repeat with other eye.

- 3) After each use and prior to recapping, if any residual gel remains on the bottle tip, simply shake the bottle downwards to remove any remaining gel. This step is important to ensure delivery of the subsequent drops.

- 4) Screw the cap to close the bottle.

## 7. CAUTIONS FOR USE

Multiple use product for single patient.

**Do not share bottles between different people to prevent cross contamination.**

Check the packaging is intact before first use: use only if the bottle and the nozzle are unopened and undamaged. Make sure that the sealed ring on the cap has not been broken.

To avoid contamination, do not touch the tip of the bottle with anything.

You must contact a healthcare professional if your symptoms get worse or do not improve.

## 8. WARNINGS

Keep out of sight and reach of children.

Do not inject and do not swallow.

After use, do not drive or use machines if your vision is blurred: wait a few minutes for your vision to clear.

Wait at least 15 minutes between using two different eye products.

**THEALOZ GEL** should be used last.

Never dilute **THEALOZ GEL** or mix it with other solutions.

After first use, if exposed to low atmospheric pressure, the bottle can leak.

## 9. SIDE EFFECTS

The followings side effects have been reported for similar devices: sticky eyes, blurred vision, allergic reactions (such as hypersensitivity, swelling of eyelid, rash), eye discharge, rare possibility of eye irritation, eye pain and ocular redness.

When you use an eye gel, you might experience disturbing symptoms such as burning sensation, stinging sensation and foreign body sensation in the eye for a short time.

If you note any side effects or unusual sensations after using this product, please contact a healthcare professional and report the information to the local distributor, to the manufacturer or to your local Health Authority (see contacts at the end of the instructions for use).

## 10. STORAGE AND DISPOSAL CONDITIONS

Store between 2 °C and 25 °C.

Do not expose your product to direct sunlight or humidity.

The product can be used until 3 months after first opening of the bottle. After this time, discard any leftover product responsibly.

Do not use after the expiry date which is marked on the bottle and on the outer box.

The expiry date refers to the last day of that month as long as the packaging is intact, and it has been stored correctly.

Dispose of the product responsibly and in accordance with local guidance (the bottle should be disposed of as household waste, the outer carton and the leaflet are recyclable).

**Date of issue and version of the instruction for use:**

**Ver. 03 – 11/2025 (CA)**

**Manufacturer:**

 Laboratoires Théa  
12 rue Louis Blériot  
63017 Clermont-Ferrand Cedex 2 France

www.laboratoires-thea.com

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CA

**Importer:**

Thea Pharma Inc.  
10 Four Seasons Place – Suite 802  
Toronto ON  
Canada, M9B 6H7

**Health authorities:**

Medical Devices Directorate  
Health Products and Food Branch  
11 Holland Avenue, Tower A2nd Floor  
Address Locator: 3002A, Ottawa, Ontario  
K1A 0K9

E-mail: meddevices-instrumentsmed@hc-sc.gc.ca

Telephone: 613 - 957 - 7285

© 2024, Laboratoires Théa. All rights reserved.

THEALOZ® is a trademark of Laboratoires Théa.

**Device identifier: THEALOZ GEL 10027-15**

**MEANING OF THE PICTOGRAMS ON THE PACKAGING**

	Manufacturer		Do not use if package is damaged and consult instructions for use
	Batch code		Caution
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
	Temperature limit		Catalogue number
	Sterilized using aseptic processing techniques		Single sterile barrier system with protective packaging outside
	Medical Device		Sterilized using steam or dry heat
	Consult instructions for use		Single patient - Multiple use

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