

## Lens Parameters

Bausch + Lomb ULTRA® ONE DAY Multifocal

<b>MATERIAL:</b>	kalifilcon A
<b>LENS MATERIAL TECHNOLOGY:</b>	Advanced MoistureSeal® Technology + ComfortFeel Technology
<b>WATER CONTENT:</b>	55%
<b>OXYGEN TRANSMISSION:</b>	134 Dk/t @ -3.00D
<b>LENS DESIGN TECHNOLOGY:</b>	3-Zone Progressive™ Design
<b>BASE CURVE:</b>	8.6 mm
<b>DIAMETER:</b>	14.2 mm
<b>CENTRE THICKNESS:</b>	0.08 mm @ -3.00D
<b>SPHERICAL POWERS:</b>	+6.00D to -10.00D in 0.25D
<b>ADD POWERS:</b>	Low: up to +1.50D, High: up to +2.50D
<b>VISIBILITY TINT:</b>	Light blue
<b>MODALITY:</b>	Daily
<b>UV PROTECTION:</b>	Yes (CLASS II)
<b>MODULUS:</b>	0.5MPa
<b>PACK SIZE:</b>	30 and 90 pack

References: † when the fitting guide was followed 1. Results from a 20-site, 3-week study of Bausch + Lomb (kalifilcon A) Daily Disposable Multifocal contact lenses on 294 habitual multifocal soft contact lens wearers. Only Bausch + Lomb ULTRA® ONE DAY Multifocal contact lenses combine a 3-Zone Progressive™ Design for effortless clarity from near to far and a complete moisture + comfort system with Advanced MoistureSeal® and ComfortFeel Technologies that offer a moisture rich environment, high Dk/t, low modulus, UV blocking. Bausch + Lomb ULTRA® ONE DAY Multifocal contact lenses deliver health through its complete system working together to support a healthy ocular surface environment, the inclusion of eye health ingredients which are retained over 16 hours and the high allowance of oxygen permeability (Dk/t=134).

Bausch + Lomb ULTRA® ONE DAY, MoistureSeal, and 3-Zone Progressive are trademarks of Bausch & Lomb Incorporated or its affiliates. Bausch & Lomb Canada.

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates. Bausch + Lomb Corporation, Vaughan, Ontario, L4K 4B4 © 2024 Bausch + Lomb. TP21150 160-8892E

**BAUSCH + LOMB**  
See better. Live better.

# FITTING GUIDE FOR Bausch + Lomb ULTRA® ONE DAY Multifocal

**STEP 1: Update** spectacle refraction and Add power

**STEP 2: Select** contact lens distance prescription based upon spherical equivalent from spectacle Rx and following Add guidance (adjusted for vertex distance if necessary)

### ADD SELECTION:

SPECTACLE Add	BOTH EYES
+0.75D to +1.50D	Low Add
+1.75D to +2.50D	High Add

### EVALUATE THE LENS FOR SUCCESS

- Allow trial lenses to equilibrate for at least 10 minutes before assessing fit and vision
- Evaluate distance and near vision binocularly in normal room illumination
- If vision at distance and near are satisfactory, dispense lenses and schedule follow-up exam within 1-2 weeks



Peers demonstrated success with the 3-Zone Progressive™ Design†1:

- Easy to fit for 99% of patients
- 87% of patients successfully fit in one visit
- 99% of patients successfully fit in two visits

## REFINE IF NEEDED

Determine eye dominance at distance by placing a +1.50 loose hand-held trial lens alternately over each eye binocularly through updated distance correction. The eye for which binocular vision is blurriest through the +1.50 is the dominant eye.

		Near Vision		Distance Vision			
		DOMINANT EYE	NON-DOMINANT EYE	DOMINANT EYE	NON-DOMINANT EYE		
If patient is wearing:	TWO LOW ADDS	Initial Lens	Low Add	Low Add	Initial Lens	Low Add	Low Add
		Refinement 1	Low Add	High Add	Refinement 1	Bausch + Lomb ULTRA® ONE DAY Sphere	Low Add
		Refinement 2: If vision is still unsatisfactory, make small changes by adding +0.25D at a time to non-dominant eye (wearing High Add lens) using hand-held lenses, and continue evaluating vision binocularly in normal room illumination. Adjust contact lens power when vision is satisfactory.			Refinement 2: If vision is still unsatisfactory, make small changes by adding -0.25D at a time to dominant eye (wearing Bausch + Lomb ULTRA® ONE DAY spherical lens) using hand-held lenses, and continue evaluating vision binocularly in normal room illumination. Adjust contact lens power when vision is satisfactory.		
	TWO HIGH ADDS	Initial Lens	High Add	High Add	Initial Lens	High Add	High Add
		Refinement 1	High Add	Add +0.25D to the non-dominant eye	Refinement 1	Low Add	High Add
		Refinement 2: If vision is still unsatisfactory, make small changes by adding +0.25D at a time to non-dominant eye using hand-held lenses, and continue evaluating vision binocularly at normal room illumination. Adjust contact lens power when vision is satisfactory.			Refinement 2: If vision is still unsatisfactory, make small changes by adding -0.25D at a time to dominant eye (wearing Low Add lens) using hand-held lenses, and continue evaluating vision binocularly in normal room illumination. Adjust contact lens power when vision is satisfactory.		

Scan to use the Bausch + Lomb Multifocal Fitting calculator!

