Zaspray®

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Zaspray®

This product is a medical device. Please read the instructions before use.

Keep this leaflet. You may need to read it again.

1. INDICATIONS

ZASPRAY is indicated to soothe, hydrate and lubricate dry eyes, especially in case of allergy.

ZASPRAY relieves itchy, dry, irritated and red eyes. Ocular use only.

2. CONTRAINDICATIONS

Do not inject and do not swallow.

Do not use **ZASPRAY** if you are allergic to any of its ingredients (see Device description).

3. DEVICE DESCRIPTION

ZASPRAY is a lubricating and lenitive eye spray containing a liposomal solution based on PER-LIP® Complex (Liposome 4% + Perilla seed extract 0.5%), sodium hyaluronate, boric acid, sodium tetraborate decahydrate, PEG 300, disodium EDTA dihydrate, sodium chloride, water.

ZASPRAY is sterile 10 ml eye spray that is preservative-free, delivered in a 10 ml bottle, equipped with the innovative APTAR® PLUS (multi-dose spray with tip seal technology), by APTAR® which avoids the need of preservatives and prevents bacterial contamination over the duration of treatment.

The absence of preservatives allows the protection of ocular tissues and also causes no reactions to users who are sensitive to preservatives.

ZASPRAY is a pleasant and easy-to-use alternative for people who have a difficult time using eye drops.

4. BENEFIT, PERFORMANCE AND MECHANISM OF ACTION

ZASPRAY relieves itchy, dry, irritated and red eyes:

- providing freshness,
- soothing symptoms of irritated eyes,
- preventing dehydration of surface of the eye,
- limiting water evaporation at the surface of the eye
- restoring the natural balance of the eye surface and stabilizing the tear film, altered by environmental stress such as allergens but also air conditioning, wind exposure, dust, pollution and by eye strain due to prolonged use of contact lenses, computer and adverse lighting conditions, etc.

ZASPRAY can be used in addition to antihistaminic therapies.

ZASPRAY is well tolerated on the ocular surface.

Sodium hyaluronate is naturally found in the eye. It improves the tear film stability and keeps the ocular surface lubricated, hydrated and protected.

The spray delivers liposomes (oily component as phospholipid) to the eye, helping to repair the lipid layer, thereby addressing the lipid instability that causes dry eyes, easing the irritation and discomfort of ocular pathologies.

Perilla is an antioxidant that helps counteracting the accumulation of oxidative stress.

The combined action of PER-LIP® complex (liposomes + perilla) and hyaluronic acid helps to restore and improve the tear film stability, keeping the eye surface more hydrated and lubricated.

5. GENERAL RECOMMENDATIONS FOR USE

Dosage: It is recommended to spray the product on closed eyelids once or twice every application, 3 to 4 times a day for 3 to 4 days consecutively. Do not exceed 30 consecutive days.

There is no particular training required to use **ZASPRAY**.

1. Wash hands thoroughly before use.
2. Remove contact lenses before applying the product.
3. Hold tightly the bottle with your thumb or forefinger on the nozzle and pay attention not to plug the nozzle hole. Hold the bottle approximately 10 cm from the eye.
4. Apply on closed eye, exerting a firm pressure on the nozzle allowing the proper nebulization of the solution.
5. Repeat on the other eye if necessary.

6. CAUTIONS FOR USE

Multiple use product for single patient.

Do not share bottles between different people to prevent cross contamination.

Check the packaging is intact before first using this product.

To avoid contamination, do not touch the nozzle to the ocular surface or anything.

You must contact a healthcare professional if your symptoms get worse or do not improve.

7. WARNINGS

Keep out of the sight and reach of children.

In case of eye irritation, stop using the product and contact your doctor.

If you note any side effects or unusual sensations after using this product, please contact a healthcare professional and report the information to the local distributor, to the manufacturer (see contacts at the end of the instructions for use).

8. STORAGE AND DISPOSAL CONDITIONS

Store the product away from heat sources, in a dry place at a temperature ranging from 9°C and 25°C.

Do not use after the expiry date, which is marked on the bottle and on the outer box.

The expiry date refers to the last day of that month as long as the packaging is intact and it has been stored correctly.

The product can be used until 90 days after first opening. After this time, discard any leftover product responsibly and in accordance with local guidance.

Date of issue and version of the instructions for use: 04/2021

Manufacturer:

NTC Srl, Via dei Gracchi, 35
20146 Milan
Italy

PES041

Distributor: Thea Pharma, Oakville, ON L6H 5V1

PER-LIP®, registered trademark of NTC.

9. MEANING OF THE PICTOGRAMS ON THE PACKAGING

	Manufacturer		Consult instruction for use
	Batch number		Single patient - Multiple use
	Medical Device		Do not use if package is damaged
	Use-by date		Caution
	Temperature limits		Sterilized using aseptic processing techniques
	Dispose of the product in accordance with local requirements		